# VERMEIREN

## 640

USER MANUAL  
MANUEL D'UTILISATION  
GEBRUIKSAANWIJZING  
GEBRAUCHSANWEISUNG  
MANUALE DI ISTRUZIONI  
MANUAL DE INSTRUCCIONES  
INSTRUKCJA OBSŁUGI  
NÁVOD K OBSLUZE

EN

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## Toilet aid 640

Thank you for your trust in the products of Vermeiren. To support you on the use of your product, this manual is offered. Please read it carefully. If you still have questions after reading this manual, do not hesitate to contact your specialist dealer. On our website <http://www.vermeiren.com/> you will always find the most recent version of the following information. Please consult this website regularly for possible updates. Visually impaired people can download the electronic version of this manual and have it read out by means of a text-to-speech software application.

### 1 Upon delivery

Unpack your product and verify if the delivery is complete. Following items should be included:

- 1 toilet aid
- Manual

Verify your product for transport damage. If you find any damages after delivery, contact the transporter.

### 2 Usage

640 is suitable as an assistive product for elderly people or people during recovery. The aid provides extra support, making it easier to sit down and get up.

#### 2.1 Altering the height

640 is height adjustable in 5 positions:

1. Loosen and remove the screw (1).
2. Move the inner tube (2) in/out of the outer tube until the desired position is reached.
3. Replace the screw and tighten firmly.
4. Repeat for the other legs. Make sure that all legs are well secured and adjusted to the same position.

### 3 Maintenance

Your product can be cleaned and disinfected with normal household products. Avoid using abrasive cleaning products.

#### 3.1 Expected lifespan

The toilet aid is designed to have an average lifespan of 5 years.

### 4 Technical specifications

<b>Brand</b>	Vermeiren
<b>Type</b>	Toilet aid
<b>Model</b>	640

Description	Dimensions
Height	645 - 755 mm
Total width	590 mm
Inner width	485 mm
Length	420 mm
Storage temperature	+5°C - +41°C

We reserve the right to introduce technical changes.  
Measurement tolerance +/- 15 mm / 1,5 kg / 1,5°.

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### All rights reserved, including translation.