VERMEIREN

260 - 261 - 262 - 264

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EN

Instructions for specialist dealer

This instruction manual is part and parcel of the product and must accompany every product sold.

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Version: B, 2016-12

All rights reserved, including translation.

No part of this manual may be reproduced in any form what so ever (print, photocopy, microfilm or any other process) without written permission of the publisher, or processed, duplicated or distributed by using electronic systems.

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INSTRUCTION MANUAL
Walking stick with more legs
260-261-262-264

1. Technical details

⚠ WARNING: Risk of unsafe settings - Use only the settings described in this manual.

Make	Vermeiren
Address	Vermeirenplein 1/15,  B-2920 Kalmthout
Type	Walking stick with more legs
Model	260
Width	224 mm
Depth	335 mm
Total height	673- 870 mm
Min height handgrip	673 mm
Max height handgrip	870 mm
Length handgrip	115 mm
Weight	1,40 kg
Maximum user weight	max. 100 kg
Model	261
Width	153 mm
Depth	273 mm
Total height	572 - 682 mm
Min height handgrip	572 mm
Max height handgrip	682 mm
Length handgrip	115 mm
Weight	1,10 kg
Maximum user weight	max. 75 kg
Model	262
Width	225 mm
Depth	328 mm
Total height	945 – 1018 mm
Min height handgrip	945 mm
Max height handgrip	1018 mm
Length handgrip	115 mm
Weight	1,65 kg
Maximum user weight	max. 100 kg
Model	264
Width	232 mm
Depth	383 mm
Total height	805 – 953 mm
Min height handgrip*	805 mm
Max height handgrip	953 mm
Length handgrip	115 mm
Weight	150 kg
Maximum user weight	max. 100 kg
We reserve the right to introduce technical changes. Measurement tolerance +/- 15 mm / 1,5 kg.	
*Do not use the two lowest height adjustment holes.	

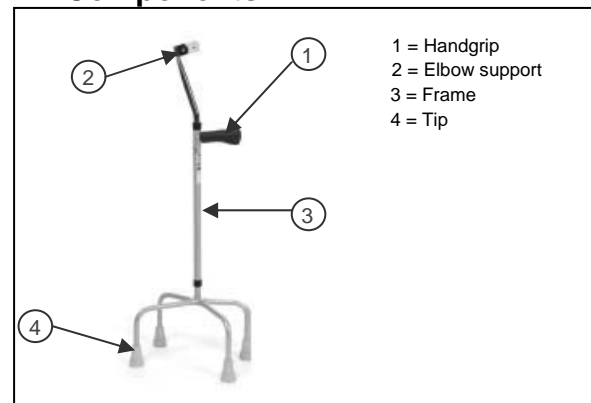
stick. Please remember that observance of our hints would keep your walking stick in an excellent condition of preservation and it will function perfectly even after years of use.

If you have any further questions, please consult your specialist dealer.

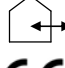
To find a service facility or specialist dealer near you, contact the nearest Vermeiren facility. A list of Vermeiren facilities can be found on the last page.

Visually impaired people can contact the dealer for the instructions for use.

2. Components

3. Explanation of symbols

-  Maximum mass
-  Indoor and outdoor use
-  CE conformity
-  Read the instruction manual before use
-  Observe the safety instructions
-  Type designation

4. Check after delivery

Unpack your product and verify if the delivery is complete. Following items should be included:

- Walking stick with more legs
- Manual

Verify your product for transport damage. If you find any damages after delivery proceed as follows:

- Contact the transporter
- Have a list made of any problems
- Contact your supplier

First of all we wish to thank you for the trust you placed in us by selecting a **VERMEIREN** product.

Before using the walking stick with more legs, read the instruction manual carefully: it will familiarise you with this product.

Refer to the Vermeiren website for the instructions for care and safety of the walking

5. Intended use

⚠ WARNING: Risk of falling - Do not use the walking stick if you have weak arm, wrist joint function or poor equilibrium.

A walking stick give the user support during walking and makes it easier to walk. These support can be useful for balance problems or pain when walking.

The walking stick can be used indoor and outdoor.

Be extra careful that you do not slip when using on wet surfaces.

However, it cannot be used by persons with a visible weakened hand-/arm function, wrist joints.

6. Height adjustment

⚠ CAUTION: Risk of falling - Check that the spring pins are good clicked back.

Before using the walking stick you must adjust these in the correct height.

Adjust the height of the walking stick according following instructions:

1. Press on the spring pin of the walking stick.
2. Adjust the walking stick at the desired height.
Ensure that the height of your wrist is at the same height of the handgrip when you leave your arms loose next your body.
3. Check that the spring pin is good clicked back.

7. Walking

Follow the advice of your doctor or physiotherapist how you must walk with the walking stick. When you have no advice, you can use the instructions below:

- Always walk upright with the walking stick right next to your body.
- Place the walking stick on the side of your unaffected leg or the side were you need support. Place your affected leg forward simultaneously with the walking stick and then your unaffected leg.

8. Walking on stairs

When there is a handrail available, you can use these as support and on the other side you can use the walking stick. Ensure that the handrail is on the healthy side of your body, if possible.

Lean on the handrail and place the walking stick on the first step. Place your unaffected leg on the first step and put your affected leg beside them.

9. Safety instructions

- Note that you use your walking stick on a good surface. (Not with too many gravel).
- Be extra careful on a wet / slippery ground.
- Before every usage, check if the walking stick is adjusted in the correct height.
- Check that the spring pins for the height adjustment are good clicked back.
- Do not place the walking stick in a curve or too far from your body to avoid risk of falling.
- Do not use the walking stick as a toy.
- Do not hold the handgrip with wet hands. If you do so, you can lose your grip and lose your balance.
- Do not take to big steps with the walking stick.
- **The max. user weight of the product is 100 kg for models 260-261-264 and 75 kg for model 261.**
- Risk of burns – Be careful when using in hot or cold environments (sunshine, extreme cold, saunas, etc.) for a sufficient amount of time and when touching - Surfaces can assume the environment temperatures.
- Take note of the instructions for care and service. The manufacturer is not liable for damage caused by improper servicing/care.

10. Maintenance

For the maintenance manual of the walking sticks refer to the Vermeiren website: www.vermeiren.com.

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CONTRACTUAL WARRANTY

We offer 5 years of warranty on standard wheelchairs, lightweight wheelchairs 4 years. Electronic wheelchairs, tricycles, beds and other products: 2 years (batteries 6 months) and multiposition wheelchairs 3 years. This warranty is limited to the replacement of defective or spare parts.

APPLICATION CONDITIONS

In order to claim its warranty, part "B" of this card has to be given to your official Vermeiren dealer. The warranty is only valid when parts are replaced by Vermeiren in Belgium.

EXCEPTIONS

This warranty is not valid in case of:

- damage due to incorrect usage of the wheelchair,
- damage during transport,
- involvement in an accident,
- a dismount, modification or repair carried outside of our company and/or official Vermeiren dealership,
- normal wear of the wheelchair,
- non-return of the warranty card

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- please return within 8 days of date of purchase or register your product at our website, <http://www.vermeiren.be/registration>

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Vermeiren Suisse S.A.