# User manual

MANUEL D'UTILISATION  
GEBRUIKSAANWIJZING  
GEBRAUCHSANWEISUNG  
MANUALE DI ISTRUZIONI  
MANUAL DE INSTRUCCIONES  
INSTRUKCJA OBSŁUGI  
NÁVOD K OBSLUZE

## Mia

**EN**

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Verze: B, 2022-07

**All rights reserved, including translation.**

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## Raised toilet seat Mia

Thank you for your trust in the products of Vermeiren. To support you on the use of your product, this manual is offered. Please read it carefully. If you still have questions after reading this manual, do not hesitate to contact your specialist dealer. On our website <http://www.vermeiren.com/> you will always find the most recent version of the following information. Please consult this website regularly for possible updates. Visually impaired people can download the electronic version of this manual and have it read out by means of a text-to-speech software application.

### 1 Upon delivery

Unpack your product and verify if the delivery is complete. Following items should be included:

- 1 raised toilet seat with cover
- 2 width adjusters
- Optional: toilet seat cushion
- Manual

Verify your product for transport damage. If you find any damages after delivery, contact the transporter.

### 2 Mounting and using the toilet seat

**CAUTION**

Risk of injuries

- Before each usage, make sure that the toilet seat is mounted securely to the toilet.
- To ensure that the toilet seat remains properly fitted, loosen and retighten the width adjustments once a week.
- Do not exceed the maximum user weight.

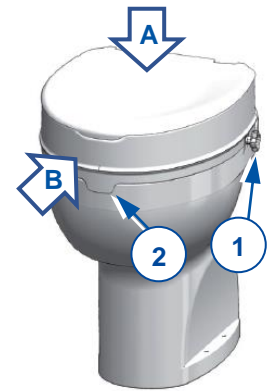
Indications and contraindications: The toilet seat is designed to raise the height of a toilet to a comfortable level, making it easier to sit down and get up, for example in case of: lower limb dysfunction due to limited or no joint mobility, reduced muscular strength in the lower limbs, damage to the lumbar spine, damage to the nervous system,... The aim of this product is to enable independent or more independent toilet use. When used accordingly, no contraindications are known.

Mia is available in 2 possible seat heights: 10 cm or 15 cm.

The maximum user weight of the raised toilet seat is 225 kg.

To install Mia:

1. Lift the toilet lid and seat. Is necessary, remove the lid and seat.
2. Insert the width adjusters (1) into place on the raised toilet seat. Turn the star knob clockwise to reduce the width a bit.
3. Place the raised toilet seat on the toilet rim [A] with the front lip (2) on the outside.
4. Push the raised toilet seat backwards [B] as far as possible.
5. Tighten the width adjusters (1) by hand.

To place the optional cushion, remove the raised seat cover.

Slowly lower yourself onto the raised toilet seat by bending your elbows, hips and knees if possible. Do not hold on to the raised seat for support while lowering yourself/standing up.

### 3 Maintenance

Your toilet seat is constructed of plastic components. Mia can be cleaned and disinfected with normal household products. Avoid using abrasive cleaning products that can cause discoloration.

#### 3.1 Expected lifespan

The raised toilet seat is designed to have an average lifespan of 5 years.

### 4 Technical specifications

<b>Brand</b>	Vermeiren
<b>Type</b>	Raised toilet seat
<b>Model</b>	Mia

Description	Dimensions
Maximum load	225 kg
Added seat height	100 mm 150 mm
Width	365 mm
Length	405 mm
Storage temperature	+5°C - +41°C

We reserve the right to introduce technical changes.  
 Measurement tolerance +/- 15 mm / 1,5 kg / 1,5°.

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## Service registration form

This product (name): .....

was inspected (I), serviced (S), repaired (R) or disinfected (D):

By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:
By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:
By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:
By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:
By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:

Vermeiren GROUP  
Vermeirenplein 1 / 15  
2920 Kalmthout  
BE

website: [www.vermeiren.com](http://www.vermeiren.com)