


COMFORT CHAIR WITH FOOT PLATE			Plan No. / Plannr.
COMFORT CHAIR MET VOETPLAAT			E000429/F
Scale / Schaal: 7:50	Drafted / Getekend	Verified / Nagezien	Confirmed / Bevestigd
Date / Datum	30/01/2008	12/06/2019	12/06/2019
Init. / Init.	SD	TT	PJ



Material / Materiaal			
No. / Nr.	Plan	Article / Artikel	Number / Aantal
1	X000319	G080200	1
3	X001016	1001682	14
4	X001119	1001554	10
5	X002480	1993793	2
6	X003827	1009400	6
7	X003903	1901170	1
8	X004034	1683600	2
9	X004038	1685700	2
10	XP00085	2990047	2
11	/	/	/
12	XP00190	2990049	2
13	Y011448	1638500	1
14	Y012124	1683482	1
15	Y012249	1667282	2
16	Y012657	1683282	1
17	Y012658	1683382	1
18	Y012851	1683582	2
19	Y013065	1678300	1
20	M12 DIN127A	1001572	4
21	---	2072782	1
21A	YP00913	1836882	1
21B	X000989	1004539	4
22	YP00915	1836982	1
23	YP00920	1837282	1
24	M6x45 DIN933/8.8	1001435	1
25	M8x25 DIN933/8.8	1001442	4
26	M8x16 DIN912/8.8	1001498	2
27	Ø16	1001598	4
28	M6 DIN985	1001680	1
30	M8x45 DIN912/8.8	1003110	2
31	M8 DIN125	1003199	20
32	M6x40	1006111	4
33	Ø4x20 DIN1481	1009584	2
34	Ø4x20	1991373	2
35	Ø8x45	1993594	1
36	---	1902259	2
37	---	1001252	2
38	---	1991628	1
39	M12x30 DIN912/8.8	1009318	4
41	M8x40	1008739	4
42	Ø22	1001604	2
43	Ø6	1992835	4
44	X003826	1009273	1
45	X004198	1701100	2
46	XP00144	1894000 (L)	1
47	XP00567	1894100 (R)	1

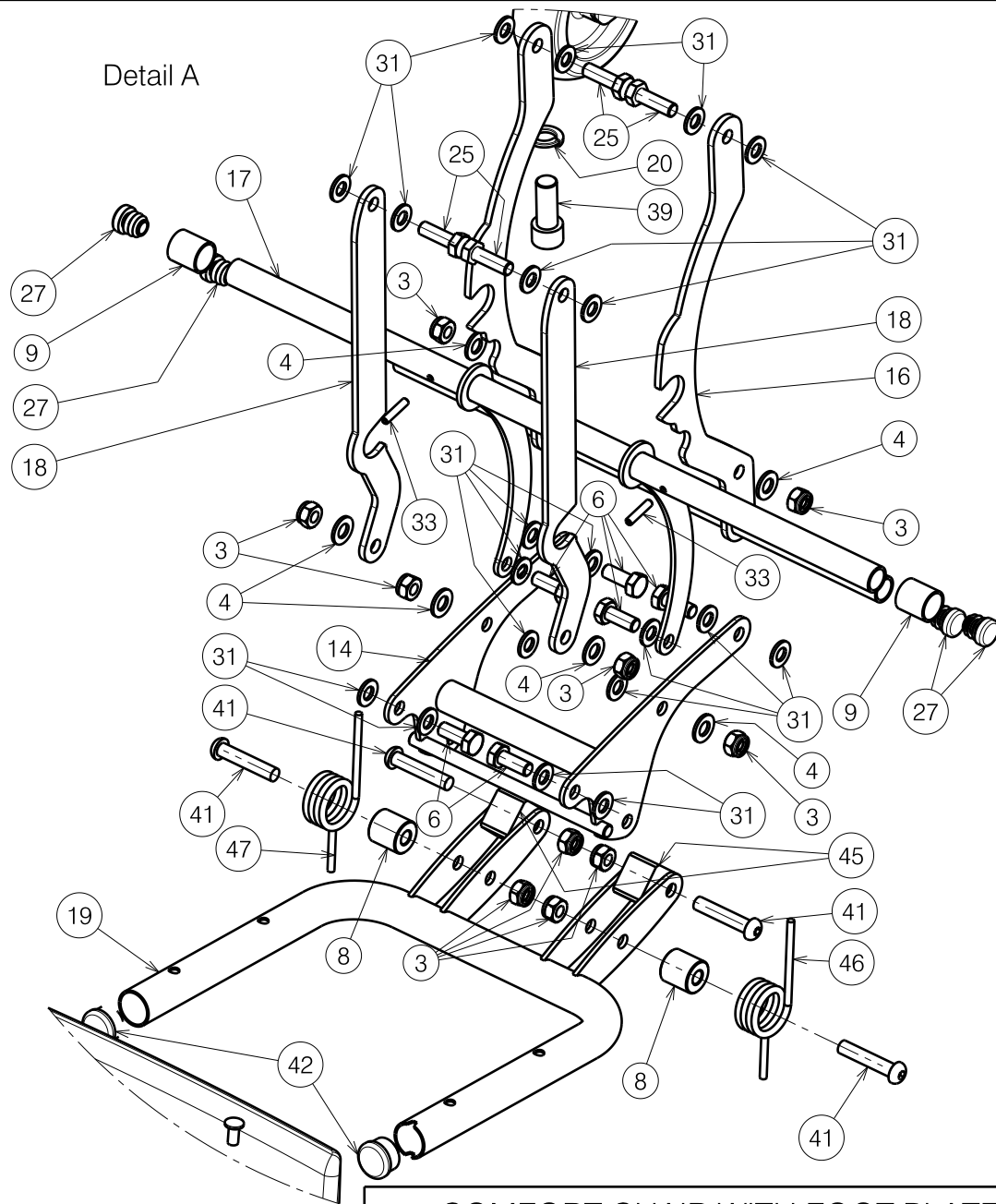

COMFORT CHAIR WITH FOOT PLATE COMFORT CHAIR MET VOETPLAAT				Plan No. / Plannr. E000429/F
Scale / Schaal: 7:50	Drafted / Getekend	Verified / Nagezien	Confirmed / Bevestigd	
Date / Datum	30/01/2008	12/06/2019	12/06/2019	
Init. / Init.	SD	TT	PJ	